TIRALATTE ELETTRICO STIMOLATTE

**ELECTRIC BREAST PUMP STIMOLATTE
TIRE-LAIT ÉLECTRIQUE STIMOLATTE
ELEKTRISCHE MILCHPUMPE STIMOLATTE
SACALECHES ELÉCTRICO STIMOLATTE
SACA-LEITE ELÉTRICO STIMOLATTE
LAKTATOR ELEKTRYCZNY STIMOLATTE
ELEKTRISCHE BORSTKOLF STIMULEREND
ΗΛΕΚΤΡΙΚΟ ΘΗΛΑΣΤΡΟ STIMOLATTE
ЕЛЕКТРИЧНИЙ МОЛОКОВІДСМОКТУВАЧ
STIMOLATTE
ЭЛЕКТРИЧЕСКИЙ МОЛОКООТСОС
STIMOLATTE**

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INSTRUCTIONS FOR USE

ELECTRIC BREAST PUMP STIMOLATTE

Please read these instructions carefully before use and keep them for future reference. Intended use and indications for the electric breast pump

SAFETY WARNINGS

WARNING!

- This appliance may be used by children 8 years of age and older and by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge, if they have received supervision or instructions on how to use the appliance safely and understand the risks involved.
- Children must not play with the device.
- Cleaning and maintenance by the user must not be carried out by children, unless they are over 8 years old and supervised.
- Keep the appliance and power cord out of reach of children under 8 years of age. The USB cable and the tube connecting the motor body to the suction kit could cause strangulation if left within reach of children.
- This product is intended for domestic use only by an adult and must be used in accordance with the instructions for use provided. Any other use is to be deemed improper and therefore dangerous. Artsana declines all responsibility for any damage caused by inappropriate, incorrect or unreasonable use of the appliance.
- The appliance is for personal use and must not be sold by the purchaser or lent to other users.
- This appliance and its components must not be exposed to direct sunlight, rain and other elements.
- Please ensure that children do not play with the appliance or its components.
- Always place the appliance only on a flat, stable surface, away from cribs and cots.
- Before connecting the mains adaptor to the mains socket, ensure that the information on the power data plate and the mains adaptor are compatible with your electricity distribution network and the corresponding socket.
- Only use certified mains adaptors. Compatible adaptor AC - DC 5.0 V, 2000mA.
- Connect the mains adaptor to a socket that is easily accessible only by an adult.
- When disconnecting the appliance from the mains electricity supply, first pull the mains adaptor plug out of the socket, or disconnect the cable from the USB plug directly, and only then disconnect the USB cable from

- the motor unit. Do not pull on the USB cable or the appliance itself if connected, to disconnect the mains adaptor from the socket.
- Do not clean or sanitise the breast pump when it is in operation, when it is receiving power from the battery or when connected to the mains. Cleaning and sanitising operations must be carried out in accordance with these instructions for use.
 - In order to avoid the proliferation of bacteria, it is advisable to feed the baby immediately after the milk has been collected, or to immediately store the collected milk in the most appropriate way.
 - Clean and sanitise the silicone cup (3), the connector (4) and the valve (5) before each use. Also clean and sanitise the feeding bottle (6) and its accessories (7)-(8).
 - Before each use, check that the membrane is not damaged: it must never be torn or cut. If it is torn or cut, do not use the appliance.
 - Any cleaning and sanitising to be performed by the user must not be carried out by children and/or people with reduced mental faculties or lacking experience or the required knowledge.
 - Do not wet the device or put it into water or other liquids. If any liquid has entered the appliance, contact our free phone number (800-188-898) or qualified technical personnel before further use.
 - If the appliance shows any sign of damage do not use it, or contact our free phone number (800-188-898) or qualified technical personnel before further use.
 - Do not attempt to open or repair the appliance if faulty or damaged. In case of faults or damage, please contact the free phone number (800-188-898). Any repair/service carried out directly by the user may lead to severe risks to the user or to any third party's safety as well as irreparably damaging the appliance and entails immediate termination of the warranty.
 - If the packaging appears to be damaged, check the state of all the components and parts of the appliance.
 - Before use, remove and dispose of all plastic bags and packaging materials and keep them out of reach of children; they may be a source of danger.
 - After removing any packaging materials, ensure that the appliance has not been damaged and proceed following the instructions provided in the paragraph about "CLEANING". If in any doubt, do not use the appliance and contact only technically qualified personnel, at the point of purchase or on our free phone number (800-188-898).
 - Make sure that the appliance is placed on a level, stable surface when used in order to avoid spillage and prevent it from falling.
 - Do not place the appliance on delicate surfaces or surfaces that may be sensitive to humidity and heat.
 - Do not use the appliance while taking a bath or shower, or when in a damp place, or close to bath-tubs, sinks, washbasins, or in any other situation in

- which there are liquids that may come into contact with the appliance.
- When using any electrical appliance, it is necessary to follow a few basic safety rules:
 - Never touch the appliance with wet or damp hands.
 - Keep the appliance and the mains adaptor away from heat sources.
 - Do not touch the appliance and the mains adaptor: if they accidentally fall into water, or accidentally come into contact with water or milk. Immediately disconnect the power supply using the master switch in your home and then disconnect the plug of the mains adaptor, if present, from the socket before touching the appliance. Do not use the appliance but contact qualified personnel or ring our free phone number (800-188-898).
 - In the event of a fault and/or malfunction of the device, switch it off immediately, unplug it from the power socket, do not attempt to tamper with it, do not use it, and contact qualified personnel or the free phone number (800-188-898).
 - The appliance must be cleaned and dried thoroughly at the end of every operating cycle.
 - In the case where the appliance is not to be used for a prolonged time, it should be thoroughly cleaned and stored in a clean and dry place away from light and heat sources and in accordance with the instructions provided concerning storage and transportation.
 - Do not dispose as normal urban waste. Collect separately as electrical/electronic equipment (WEEE) as required by Directive 2012/19/EU or by the laws in force in the countries where it is sold.
 - To avoid any risk of fire or electric shock, ensure that the USB cable does not pass under any carpets or close to radiators, stoves and heaters.
 - Operate the appliance at a temperature range between +5°C and +40°C. Use outside these temperature limits, and in particular above the upper limit, may cause the appliance to overheat with consequent damage to it and/or risk of burns or access to live parts of it by the user.
 - The appliance must not be used continuously for more than 30 minutes. When you have finished using it, the appliance must be switched off for at least 30 minutes. If the appliance is used for longer than the permitted maximum operating time, even though it has a system to automatically switch it off after 30 minutes, this may cause the appliance to overheat with consequent damage to it and/or risk of burns or access to live parts of it by the user.
 - product features might be subjected to change without notice.
 - If you have any doubts on interpretation of the contents of this instruction booklet, contact your dealer or call our free phone number (800-188-898).
 - This appliance must only be supplied at SELV.
 - This appliance contains batteries that are non-replaceable.

LITHIUM BATTERY WARNINGS

This product uses a lithium-ion battery as a power supply

 If not in use for a long period, keep the unit in a cool, dry place and charge the battery every 3 months.

INDEX OF SYMBOLS

	Product catalogue reference code		Direct Current
	WARNING!		Complies with European Directive 2012/19/EU
	For indoor use only		Stand by (push)
	The appliance complies with the essential requirements of applicable EC Directives		Serial number

THE BREAST PUMP AND BREASTFEEDING

Breastfeeding is a very important stage in a baby's and in a mother's life. For babies, latching on and sucking means not only satisfying a primary physiological need, but also, and more importantly, a precious moment in their relationship with their mothers, full of emotions and tactile, auditory, taste, and olfactory sensations. Satisfying this need gives a great feeling of tranquillity and well-being to both the mother and the child. Breast milk contains all the nutrients required to meet infants' nutritional and psychological needs at least for the first 6 months of their lives. Chicco's electric breast pump helps extend breastfeeding, in the event of excess milk or when you cannot always be with your baby. In such cases you can extract milk, store it, and give it to baby when required.

LIST OF COMPONENTS

- 1- Membrane
- 2- Membrane cover
- 3- Silicone cup
- 4- Connector
- 5- Valve
- 6- NaturalFeeling feeding bottle adaptor
- 7- NaturalFeeling feeding bottle
- 8- Om teat with cap and ring
- 9- Airtight cap
- 10- Base
- 11- Motor unit
- 12- USB Cable
- 13- Silicone tube for breast pump use
- 14- Silicon cover
- 15- Mode selection
- 16- ON/OFF
- 17- Raise level
- 18- Decrease level
- 19- LED light
- 20- Battery level
- 21- Timer
- 22- Stimulation: 10 levels
- 23- Standard extraction: 10 levels
- 24- Intense extraction: 10 levels

SET-UP

- Before using the breast pump for the first time, thoroughly wash all components, with the exception of the motor unit (11), the USB cable (12) and the silicone tube (13), as described under "CLEANING".
- Then assemble the breast pump as follows:
- Push the valve (5) down into the circular duct in the base of the connector (4). (Fig.1) Note: fit the valve preferably with the tab positioned inside the connector.
- Push the outer edge to prevent damaging the valve and make sure

that both closing edges of the assembled valve are joined together. If not, reposition the valve, making sure that the two edges are correctly aligned.

- Assemble the silicone membrane (1) with the membrane cover (2) and insert them onto the connector (4) (Fig.2)
- Insert the silicone cup (3) tightly into the connector (Fig.3) so that it does not vent during use.
- Screw the NaturalFeeling feeding bottle (7) to the adapter (6). (Fig.4a)
- Screw the NaturalFeeling bottle with the adapter to the connector (4). (Fig.4b)
- The Electric Breast Pump can also be used with other Chicco feeding bottles by removing the adaptor provided (6).
- Use the practical base (10) to lay the assembled product on a flat surface and improve its stability.
- The breast pump can be used via the USB cable (12) supplied by connecting it to the mains using a compatible 5V 2A mains adaptor or on battery power. To connect the breast pump to the power and recharge it, plug the USB cable into the socket on the motor unit (Fig.5), attach it to a compatible 5V 2A adaptor and then connect it to the power.

For faster charging we recommend recharging the breast pump when not in use.

WARNING: only use the breast pump while charging with compatible 5V 2A mains adapters, otherwise the breast pump could be damaged beyond repair.

WARNING: in case of prolonged non-use, recharge the battery at least once every 3 months.

- Connect the silicone tube (13) to the membrane cover (2) (Fig.6)
- Open the silicone cover (14), rotate it and insert the rigid connector for the silicone tube (13) into the slot provided on the motor unit (11) (Fig.7)
- Take care that you do not lose the connector, if it is lost you may not be able to use the appliance.

USING THE BREAST PUMP

- Before each use, sanitise all components, with the exception of the motor unit (11), the USB cable (12), and the silicone tube (13), using cold sterilising liquids (according to the manufacturer's instructions), or hot (steam) using the appropriate equipment, or by boiling (5 minutes), allowing all hot-sanitised components to cool completely before use.
- Wash your hands and assemble the breast pump as shown in the PREPARATION paragraph.
- Sit down comfortably and relax. Note: while using the breast pump, to ensure correct operation of the appliance, take care not to cover the hole on the left-hand side of the connector (4) with your hand. Note: when switched on, the breast pump always starts in stimulation mode to facilitate milk extraction, using the factory

settings (level 1). The stimulation indicator (22) will appear on the display.

- The cup (3) can be rotated manually, choosing which areas to stimulate and which to protect. The stimulation zone is represented by the pink petals, the protection zone by the transparent part. Once the position has been chosen, make sure that the cup is still fully inserted in the connector (4) to avoid venting.
- Fit the silicone cup (3) over your breast ensuring that the nipple is centred properly.
- Switch on the breast pump by pressing the on/off button (16) on the motor unit (11), it will automatically switches on in stimulation mode (22). The display shows the battery charge status (20) and the timer (21) counting the minutes of operation.
- After approximately 2 minutes of stimulation, the appliance automatically switches to extraction mode using the factory settings (level 1). The standard extraction indicator (23) will appear on the display.
- You can stop the stimulation stage and switch to the extraction stage by pressing once on the mode selection button (15). To switch to intensive extraction mode (24), press the mode selection button (15) twice.

By repeatedly pressing the mode selection button (15), 3 different modes can be selected:

4- Stimulation (22)

5- Standard extraction (23)

6- Intense extraction (24)

- Use the "+" (17) and "-" (18) buttons on the display to adjust the intensity of stimulation and extraction.
- If the level of extraction/stimulation chosen causes discomfort or pain, reduce it to a suitable level by pressing the "-" button (18).
- **WARNING:** We recommend starting to use Stimolatte from the standard extraction mode and from the lower levels. Gradually increase the intensity of extraction until you reach your own level of comfort and pressure on breast.
- **Note:** To avoid all danger deriving from spilt liquid (milk), we recommend that you never fill the feeding bottle beyond the highest level indicated on its graduated scale.
- Should you wish to change the rotation of the cup, pause the breast pump by briefly pressing the on/off button (16), detach the cup from the breast, rotate it to the desired position, reposition on the breast and briefly press the on/off button (16) again to restart it.
- To switch the breast pump off, press and hold the on/off button (16).
- The motor base is equipped with a night light that can be switched on with the motor on or off by pressing the LED light icon (19) for about 3 seconds. The LED light turns off automatically after 30 minutes.

CLEANING

- Before carrying out any cleaning operations, make sure that the product is not connected to the mains and that it is switched off.
- Separate all the components from the breast pump. **Note:** remove the valve (5) by pulling gently.
- **WARNING:** Make sure that the valve is always removed in a gentle way; do not use any tool for cleaning it because it may be damaged, making the breast pump impossible to use.
- Thoroughly wash all components, except the motor unit (11) the USB cable (12) and the silicone tube (13) with warm water and dishwashing liquid, rinse thoroughly and dry thoroughly with a soft, clean cloth.
- Do not use abrasive products to clean the breast pump and its components.
- Clean the motor unit with a soft, clean and dry cloth. **Warning!** Do not wet the appliance or put it into water or other liquids.
- The silicone tube (13) may be rinsed and suitably dried.
- The following can be washed in the dishwasher using a delicate wash cycle: the silicone cup (3), the membrane cover (2), the connector (4), the NaturalFeeling bottle adaptor (6), the feeding bottle (7) and all its accessories (8-9-10). This, more aggressive, type of washing may discolour or darken the product over time and degrade the material.

- Also clean and sanitise the feeding bottle (7) and its accessories (8)-(9)-(10).
- After washing, always store the product in a clean, dry place.

Fault	Cause	Solution
The breast pump does not pump or pumps very little	Valve not assembled or not properly assembled.	Assemble the valve correctly.
	Silicone tube incorrectly assembled to the membrane cover.	Check that you have performed the procedure described in the "PREPARATION" paragraph correctly. Make sure that the silicone tube is firmly connected to both the motor unit and the membrane cover. Check that the tube is not damaged. Do not twist the tube during milk expression.
	The silicone cup does not adhere well to the breast or is not properly inserted into the connector.	Fit the cup over your breast, taking care that your nipple is properly centred. Press the silicone cup firmly into the connector.
	Suction level not correctly adjusted.	Change the suction level until you achieve more comfortable stimulation and suction by pressing the "+" or "-" buttons.
	Low or insufficiently charged battery.	Recharge with USB cable.
	Membrane and/or cover not assembled or poorly assembled.	Properly assemble the membrane in its housing and the cover over it.
Milk expression is painful.	Suction level too high.	Change the suction level until you achieve more comfortable stimulation and suction by pressing the "-" button.
The breast pump does not switch on.	Low or insufficiently charged battery.	Check that mains power is present once the product is plugged in.
	Incorrect connections.	Check all the connections.
	Low or insufficiently charged battery.	Recharge with USB cable.

TECHNICAL SPECIFICATIONS

- Maximum vacuum level in extraction mode: -0.35 ± 0.03 bar
- Type of appliance: intermittent operation; 30 minutes ON - 30 minutes OFF
- Motor body weight: 294 g batteries included
- Dimensions: 90x130x85h mm
- Made in China

POWER SUPPLY:

Rechargeable Li-ion battery 3.7V 2000mAh/power supply 5V DC 2A

DECLARATION OF EC COMPLIANCE

The Chicco brand Electric Breast Pump **[REF] 000091996000000 Mod. N0003** complies with all the essential requirements and other provisions indicated in the EU Directives 2014/30/EU (electromagnetic compatibility), 2014/35/EU (Low Voltage), and 2011/65/EU (RoHS). A copy of the EC declaration can be requested from the manufacturer: Artsana S.p.A via Saldarini Catelli, 1 - 22070 Grandate (Como) Italy.

The Electric Breast Pump [REF] 000091996000000 Mod. N0003 complies with EU Directive 2023/1542.

The crossed bin symbol on the batteries or product pack indicates that, at the end of their life, they must not be disposed of as urban refuse. They must be disposed of separately from domestic waste, either by taking them to a separate waste disposal site for batteries or by returning them to your dealer when you buy similar rechargeable or non-rechargeable batteries. The chemical symbols Hg, Cd, Pb, printed under the crossed bin symbol, indicate the type of substance contained in the batteries: Hg=Mercury, Cd=Cadmium, Pb=Lead. The user is responsible for taking the batteries to special waste disposal sites at the end of their life to allow for their processing and recycling. Correctly separating used batteries for subsequent recycling, treatment and environmentally sound disposal, helps avoid possible negative effects on the environment and health and promotes the recycling of the substances contained within the batteries. Illegal disposal of the product by the user will damage the environment and human health. For further information regarding the waste disposal services available, contact your local waste disposal agency or the shop where you bought the appliance.

The Electric Breast Pump [REF] 000091996000000 Mod. N0003 complies with Directive 2012/19/EU.

The crossed bin symbol on the appliance indicates that the product, at the end of its life, must be disposed of separately from domestic waste, either by taking it to a separate waste disposal site for electric and electronic appliances or by returning it to your dealer when you buy another similar appliance. The user is responsible for taking the appliance to a special waste disposal site at the end of its life. If the unwanted appliance is collected correctly as separate waste, it can be recycled, treated and disposed of ecologically; this avoids a negative impact on both the environment and health, and contributes towards the recycling of the product's materials. Illegal disposal of the product by the user will result in administrative sanctions provided by law. For further information regarding the waste disposal services available, contact your local waste disposal agency or the shop where you bought the appliance.

WARRANTY

The product is guaranteed against any conformity defect in normal conditions of use as provided for by the instructions for use. The warranty shall not therefore apply in the case of damage caused by improper use, wear or accidental events. For the duration of warranty on conformity defects please refer to the specific provisions of national laws

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Artsana S.p.A – via Saldarini Catelli, 1 - 22070 Grandate (Como) – Italy